

AirC Compressor Coil Replacement



Replacement Coil



Coil Location on scissors

1. Use the foot pedal to elevate the table
2. Unplug the table
3. Remove the screws holding the lower section of skirting to the base.
4. Lift the skirt to allow access to the air compressor area shown in photo above.
The skirt can be propped up or held up with tape.
5. Use a 9/16" wrench to remove the nut on top of the coil and remove coil.
6. Switch the wires to the new coil
7. Locate the new coil over the stem and replace nut.
8. Reattach the skirt

PROPRIETARY AND CONFIDENTIAL

THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF HILL LABORATORIES COMPANY. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF HILL LABORATORIES IS PROHIBITED.

DATE

2/2/05

HILL LABORATORIES

3 Bacton Hill Rd., P.O.Box 2028
Frazer, PA 19355
Ph.(610)644-2867 Fx.(610)647-6297

SHEET 1 OF 1