


# Air Flex - Modify Original Cushions for New Style Drop Rods

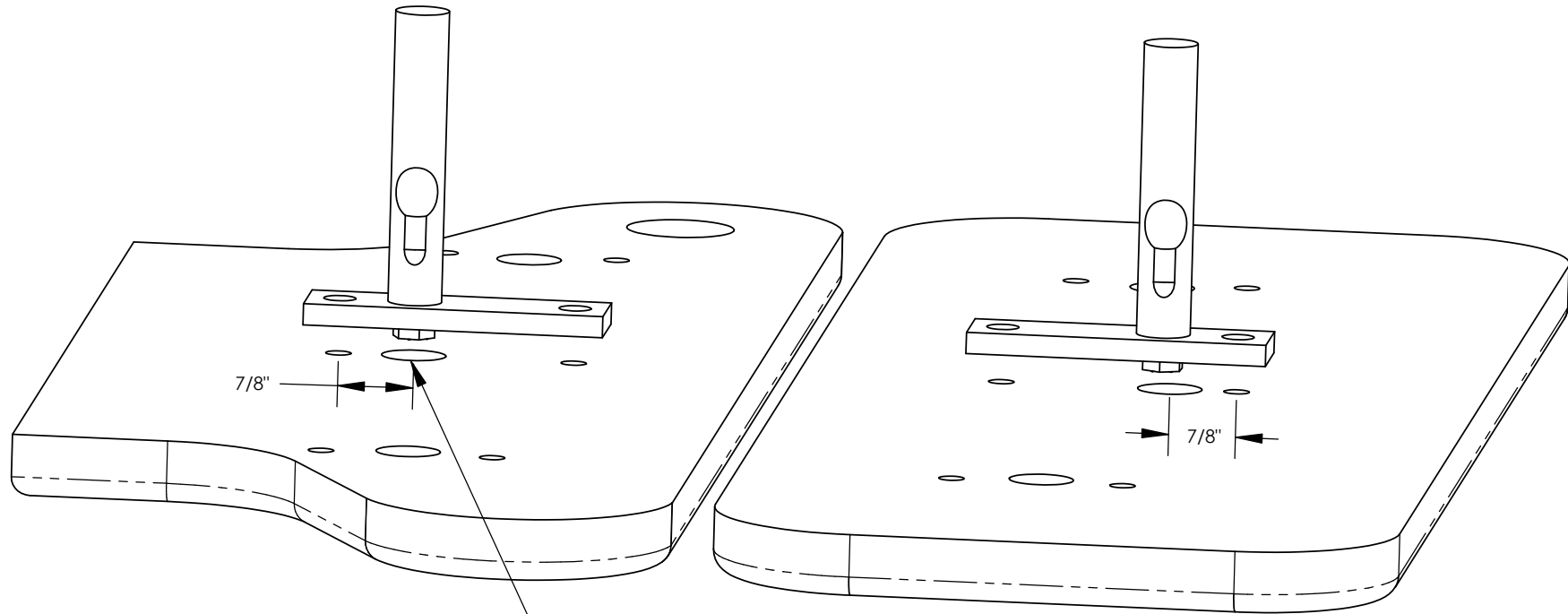

A recess must be created in the underside of each cushion at the location of the drop rod in order for the new style drop rod to be bolted to the cushion.

A minimum 5/8" diameter hole is recommended to accept the hex head of the bolt. The hole can be as shallow as 3/8". Use caution, the plywood is 3/4" thick. A spade bit is recommended.

The new hole is located 7/8" to the closer of the two threaded holes.

## PROPRIETARY AND CONFIDENTIAL

THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF HILL LABORATORIES COMPANY. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF HILL LABORATORIES IS PROHIBITED.

DATE

6/17/03

HILL LABORATORIES

3 Bacton Hill Rd., P.O.Box 2028  
Frazer, PA 19355  
Ph.(610)644-2867 Fx.(610)647-6297

SHEET 1 OF 1