


Removing & Reattaching a Mat Table Cushion

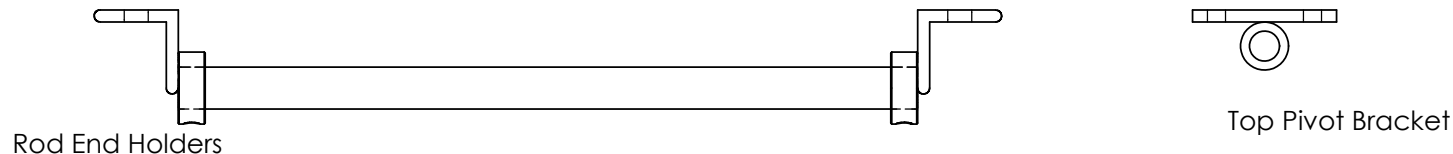

1. Use the foot pedal to elevate the table.
2. Mark the current upper skirt location by tracing the outline of all the wood strips where they contact the underside of the cushion. Also mark on the underside of the cushion a direction reference note such as "P C" (for power cord) at that end of the cushion so that repositioning the cushion will be faster.
3. Remove the screws that secure the wood strips to the cushion and allow the skirt to rest on the base.
4. Trace each of the six (6) brackets that mount the cushion to the scissors. This will assure faster assembly by saving realignment time.
5. Remove the screws holding the six brackets. Save the washers.
6. Rest the cushion on a flat surface to prevent damage to the upholstery.
7. Use the pedal to lower the height of the scissors for easier transport through the building.

REATTACH

8. Use the pedal to raise the height of the scissors.
9. Lay the cushion on top of the scissor brackets.
10. Position the cushion by locating each bracket within it's pencil tracing and insert all screws. Include the washers on the Rod End Holder screws.
11. Secure the Rod End Holders.
12. Secure the Pivot Brackets.
13. Run the table up and down several times to verify alignment.
14. Lift and locate the outer skirt to its original loaction and replace the screws in their original holes. This is important to prevent the two sections of skirt from rubbing as the table raises and lowers.

PROPRIETARY AND CONFIDENTIAL

THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF HILL LABORATORIES COMPANY. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF HILL LABORATORIES IS PROHIBITED.

DATE 1/18/05

HILL LABORATORIES

3 Bacton Hill Rd., P.O.Box 2028
Frazer, PA 19355
Ph.(610)644-2867 Fx.(610)647-6297

SHEET 1 OF 1