


REPLACING THE ROCKER FOOT PEDAL SWITCH

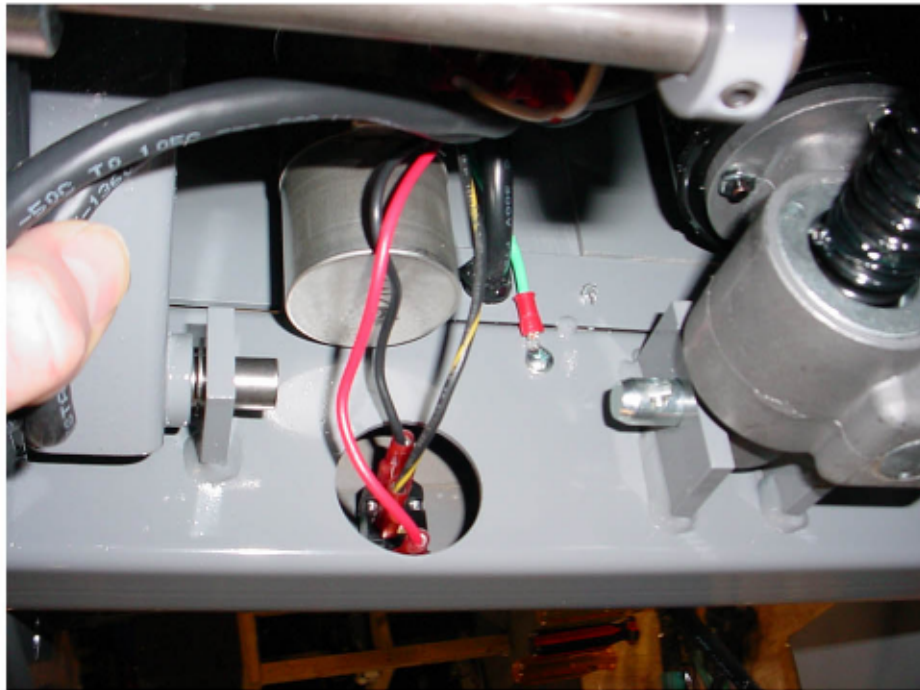

1. Unplug the table
2. Turn the table on its side to access the switch.
Be carefully not to bend the pedal as you rest the table on the floor.
3. Locate the toggle switch inside the cross member of the base. Refer to the photo at left.
4. Take note of the wire colors then unplug the wires from the switch.
5. Plug the same wire colors onto same terminals of the new switch.



6. Remove the old switch from the base by loosening the retaining nut from outside of the base cross member as shown to the left.
7. Insert the new switch, stem first into the hole in the cross member, from the inside of the base.
8. Stand the table back on its foot pads and plug in the power cord.

PROPRIETARY AND CONFIDENTIAL

THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF HILL LABORATORIES COMPANY. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF HILL LABORATORIES IS PROHIBITED.

DRAWN

NAME

B Rickards

DATE

1/17/05

HILL LABORATORIES

3 Bacton Hill Rd., P.O.Box 2028
Frazer, PA 19355
Ph.(610)644-2867 Fx.(610)647-6297

SHEET 1 OF 1